Sugi®

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Stand der Information 12.09.2023

Sugi®

Sterile absorbent material for use in the field of surgery (single use)

- Eyespear blunt tip**
- Eyespear pointed tip**
- Eyedrain**
- Sponge Strips rectangular**
- Sponge Strips rhomboidal**
- Sponge Points**
- DARC**

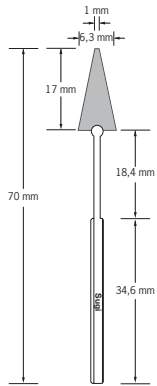
Manufacturer:
QUESTALPHA GmbH & Co. KG
Im Heerfeld 7
35713 Eschenburg, Germany
www.questalphi.com


QUESTALPHA
For Medical and Life Science

Distributed in the U.S. by:
Kettenbach LP
62-64 Enter Lane
Islandia, NY 11749, USA


Made in Germany
37244/0925

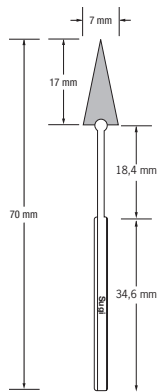Sugi Eyespear
(Absorbent swabs, mounted, sterile, blunt tip)
200 mounted swabs,
20 sachets à 10
REF 30901

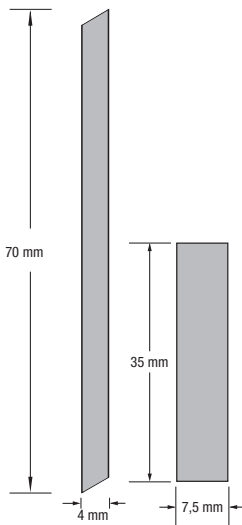
Sugi Eyespear
(Absorbent swabs, mounted, sterile, pointed tip)
200 mounted swabs,
20 sachets à 10
REF 30601

Sugi Sponge Strips
(absorbent strips, sterile, rhomboidal)
100 absorbent strips,
10 sachets à 10
REF 31301

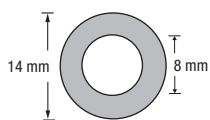Sugi Sponge Strips
(absorbent strips, sterile, rectangular)
200 absorbent strips,
20 sachets à 10
REF 31002
500 absorbent strips,
50 sachets à 10
REF 31005

Sugi DARC (sterile)
Drug Application Ring for the Cornea acc. to Prof. Spitznas
14 x 8 mm
25 sachets à 1
REF 30652
10 x 5 mm
25 sachets à 1
REF 30653

Composition

Cellulose
Regenerated cellulose
The composition relates to dry material.

Properties

- Sugi absorbent materials are manufactured from high grade material, based on cellulose.
- The absorbent material undergoes soft/elastic expansion immediately upon contacting moisture.
- The absorptive capacity is approximately 17 times its own weight.
- Sugi absorbent materials absorb rapidly.
- The absorptive capacity is reduced when absorbing nonhydrous liquids.

Indications

- Ophthalmologic and dental interventions

Range of Applications

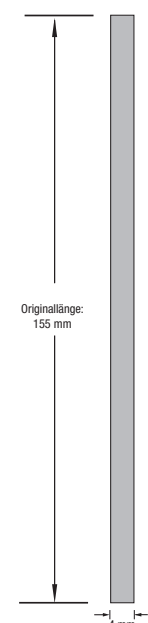
- For quick absorption of blood and secretion.
- For use as a carrier medium for liquids such as
- mydriatics, miotics (DARC, Sponge Strips).
- For keeping tissue moist (DARC, Sponge Strips).
- For draining liquids continuously (Eyedrain).

Sugi absorbent materials are suitable for dabbing those areas where care must be taken not to harm the tissue, e.g. during ophthalmic surgery.


Also, as Sugi absorbent materials retain fluids, they keep the area moist and provide mechanical protection, e.g. during microsurgery.

Contraindications and Incompatibilities

None known.

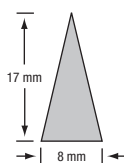Sugi Eyedrain (sterile)
20 x 1 piece,
in sachets
REF 30622

Sugi Sponge Points
(absorbent swabs, sterile, triangular)
200 swabs
20 x 10 pieces,
in sachets
REF 31602
500 swabs
50 x 10 pieces,
in sachets
REF 31604

Caution!

- Sugi absorbent products must be disposed of in the same way as dressing material and must not be reused, as there is a risk of infection and the function and safety of the product is no longer guaranteed.
- Prior to use, visually inspect the sterile barrier system for damage and do not use it if damaged or already opened.
- The shown Sugi absorbent materials are radiolucent.
- Sugi absorbent materials are to be used by medical personnel qualified to perform or assist in surgery.

Intended patient group

Patients requiring the absorption of body fluids or topical administration of liquids

Legend

Keep away from sunlight	
Use-by date	
Sterilized using ethylene oxide	STERILE EO
Do not re-use	
Keep dry	
Batch Code	LOT
Dual system (Germany only)	
Catalogue number	REF
Consult instructions for use	
Do not use if package is damaged	
Temperature limit	23° ± 2°
Medical device	MD
Date of manufacture	
Do not re-sterilize	

Limitation of liability

Except where prohibited by law, QUESTALPHA GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the legal basis of the claim, including warranty, contract, negligence, or strict liability.

The information provided for QUESTALPHA products is based on comprehensive research and experience in application technology.

Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use.

In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

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Eyespear blunt tip Eyespear pointed tip Eyedrain Sponge Strips rectangular Sponge Strips rhomboidal Sponge Points Sponge Points DARC

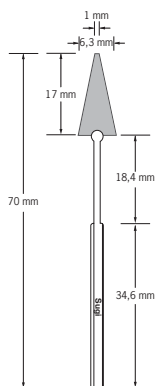
Manufacturer:
QUESTALPHA GmbH & Co. KG
Im Heerfeld 7
35713 Eschenburg, Germany
www.questalpha.com

QUESTALPHA
For Medical and Life Science

Distributed in the U.S. by:
Kettenbach LP
62-64 Enter Lane
Islandia, NY 11749, USA

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