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Stand der Information 5. Februar 2021

Sterile, disposable absorbent material.

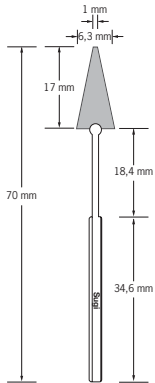
Eyepesear blunt tip
Eyepesear pointed tip
Eyedrain
Sponge Strips rectangular
Sponge Strips rhomboidal
Sponge Points
DARC

Manufacturer:
QUESTALPHA GmbH & Co. KG
 Im Heerfeld 7
 35713 Eschenburg, Germany
 www.questalpha.com


Distributed in the U.S. by:
Kettenbach LP
 62-64 Enter Lane
 Islandia, NY 11749, USA

Made in Germany
 37244/0221


Sugi absorbent swabs, mounted, sterile (blunt tip)

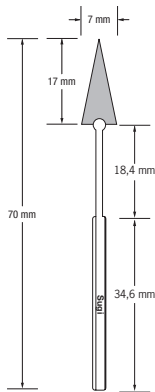200 mounted swabs
 20 x 10 pieces,
 in sachets
 REF 30901

Sugi absorbent swabs, mounted, sterile (wedge-shaped)

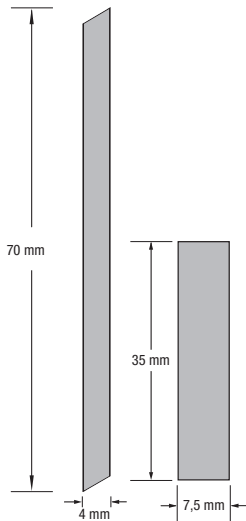200 mounted swabs
 20 x 10 pieces,
 in sachets
 REF 30601

Sugi absorbent strips, sterile (rhomboidal)

100 absorbent strips
 10 x 10 pieces,
 in sachets
 REF 31301

Sugi absorbent strips, sterile (rectangular)

200 absorbent strips
 20 x 10 pieces,
 in sachets
 REF 31002

500 absorbent strips
 REF 31003

Sugi DARC steril

Drug Application Ring for the Cornea acc. to Prof. Spitznas

14 x 8 mm,
 25 x 1 piece, in sachets
 REF 30652

10 x 5 mm,
 25 x 1 piece, in sachets
 REF 30653

Composition

Cellulose
 Regenerated cellulose

The composition relates to dry material.

Properties

- Sugi absorbent materials are manufactured from high-grade material, based on cellulose.
- The absorbent material undergoes soft/elastic expansion immediately upon contacting moisture.
- The absorptive capacity is approximately 17 times its own weight.
- Sugi absorbent materials absorb rapidly.
- The absorptive capacity is reduced when absorbing non-hydrous liquids.

Range of Applications

- Ophthalmology
- Microsurgery
- Dentistry

Sugi absorbent materials are especially suitable for dabbing those areas where special care must be taken not to harm the tissue, e.g. during ophthalmic surgery.

Also, as Sugi absorbent materials retain fluids, they keep the area moist and provide mechanical protection, e.g. during microsurgery.

Application

For quick absorption of blood and secretion.

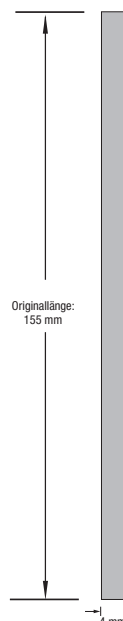
For use as a carrier medium for liquids such as mydriatics, miotics (DARC, absorbent strips).

For keeping tissue moist (DARC, absorbent strips).

For draining liquids continuously (Eyedrain).


Contraindications and Incompatibility

None known.

Sugi Eyedrain sterile

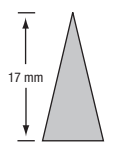20 x 1 piece,
 in sachets
 REF 30622

Sugi absorbent swabs, sterile (wedge-shaped)

200 swabs
 20 x 10 pieces,
 in sachets
 REF 31602

500 swabs
 REF 31603

Caution!

- Sugi absorbent materials should to be disposed of and must not be reused, as there is a risk of infection and the function and safety of the product are no longer guaranteed.
- Do not use sterile products if the sterile packaging is damaged or has already been opened.
- The shown Sugi absorbent materials are radiolucent.
- **Sugi absorbent materials are for use by qualified medical personnel only.**

Legend

Keep away from sunlight	
Use-by date	
Sterilized using ethylene oxide	STERILE EO
Do not re-use	
Keep dry	
Batch Code	LOT
Dual system (Germany only)	
Catalogue number	REF
Consult instructions for use	
Do not use if package is damaged	
Temperature limit	
Medizinprodukt	MD
Date of manufacture	

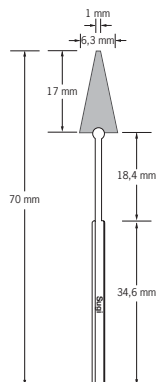
Limitation of liability

Except where prohibited by law, QUESTALPHA GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for QUESTALPHA products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

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